



### INSTRUCTIONS FOR USE

1. Insert the slide inside the plastic or metal clip, making sure the slide's label frosted side is on top and facing the funnel
2.
  - a) Place the appropriate filter over the slide
  - b) Fit the matching funnel on top of the filter
  - c) For the pre-assembled funnel with filter, place the unit over the slide
3. Secure the funnel chamber, filter and slide using the metal or plastic clip
4. Fill the funnel with the sample
5. Place the cap on the funnel to avoid spillage
6. Load the funnel into the centrifuge by following the equipment operator's guide

### Product Description:

- White filter card for sample volumes of up to 0.5 ml
- Brown filter card for slower absorption of smaller volumes up to 0.4 ml, ideal for scant specimen