



701 Hemlock Road - Morgantown, PA 19543, U.S.A. - Tel. (610) 524-5810 - Fax (610) 901-3046 - www.azersci.com

Alcian Blue (pH 2.5) Stain Kit

Description: The Alcian Blue (pH 2.5) Stain Kit is intended for use in the histological visualization of sulfated and carboxylated acid mucopolysaccharides and sulfated and carboxylated sialomucins (glycoproteins).

Acidic Sulfated Mucosubstances: Blue
 Hyaluronic Acid: Blue
 Sialomucins: Blue
 Nuclei: Red
 Background: Pink



Uses/Limitations: For In-Vitro Diagnostic use only.
 Histological applications.
 Do not use past expiration date.
 Use caution when handling these reagents.

Control Tissue: Small Intestine
 Appendix
 Colon

U
Availability/Contents:

<u>Kit Contents</u>	<u>Volume</u>	<u>Storage Conditions</u>
Alcian Blue, pH 2.5	250 ml	Room Temperature
Nuclear Fast Red Solution (Enhanced Stability)	250 ml	Room Temperature
Acetic Acid Solution	500 ml	Room Temperature

Precautions: Avoid contact with skin and eyes.
 Harmful if swallowed.
 Follow all Federal, State, and local regulations regarding disposal.
 Use in chemical fume hood whenever possible.

Storage: 18° C 25° C

Store Components at Room Temperature.


701 Hemlock Road - Morgantown, PA 19543, U.S.A. - Tel. (610) 524-5810 - Fax (610) 901-3046 - www.azersci.com

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Incubate slide in Acetic Acid solution for 3 minutes.
3. Incubate slide in Alcian Blue (pH 2.5) solution for 30 minutes at room temperature or 15 minutes at 37° C.
4. If desired, rinse slide briefly in Acetic Acid solution to remove excess Alcian Blue.
5. Rinse for 2 minutes in running tap water followed by 2 changes of distilled water.
6. Stain slide in Nuclear Fast Red Solution for 5 minutes.
7. Rinse for 2 minutes in running tap water followed by 2 changes of distilled water.
8. Dehydrate through graded alcohols.
9. Clear, and mount in synthetic resin.

References:

1. Lillie, R.D. 1977, H.J. Conn's Biological Stains, 9th Edition. Williams & Wilkins, Baltimore. Pages 452-455.
2. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. Battelle Press, Columbus, OH. Pages 172-173.
3. Churukian, C.J., 1989, Manual of Special Stains Laboratory, 4th Edition. University of Rochester, Rochester, New York. Pages 55-56.
4. Carson, F.L., 1996, Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL. Pages 117-121.

Storage: 18° C  25° C

**Store Components at Room
Temperature.**